N5B Nasal Mask User Manual

N5B Nasal Mask

Thank you for choosing BMC's N5B Nasal Mask. The Nasal Mask is comfortable, flexible and quiet. It is designed to minimize contact with your face, thus ensuring that you feel comfortable during therapy. This user manual provides you with the information you need for the correct use of your mask.

Intended Use

The N5B Nasal Mask channels airflow non-invasively to a patient from a positive airway pressure device such as a continuous positive airway pressure (CPAP) or Bi-level system.

The N5B Nasal Mask is:

- To be used by adult patients (> 66 lb / 30 kg) for whom positive airway pressure therapy has been prescribed.
- Intended for single-patient re-use in the home environment and multi-patient re-use in the hospital / institutional environment.

CAUTION: In the US, Federal law restricts the sale of this device to sale by or on the order of a physician.

Contraindications

This mask should be used with caution for patients with following conditions:


- Facial deformity.
- Injured facial skin which will contact with the mask during use.

Before Using the Mask


⚠️ WARNINGS

- The vent holes must be kept clear.
- This mask should only be used with CPAP or bilevel devices recommended by a physician or respiratory therapist.
- The mask should not be used unless the device is turned on. Once the mask is fitted, ensure the device is blowing air.
- Explanation:** CPAP and bilevel devices are intended to be used with special masks (or connectors) which have vent holes to allow continuous flow of air out of the mask. When the device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask vent holes. However, when the device is not operating, insufficient fresh air will be provided through the mask, and the exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This applies to most models of CPAP or bilevel devices.
- Not applicable for silicone-allergy user.
- This mask should not be used on patients who are uncooperative, obtunded, unresponsive, or unable to remove the mask.
- Follow all precautions when using supplemental oxygen.
- Oxygen flow must be turned off when the CPAP or bilevel device is not operating, so that unused oxygen does not accumulate within the device enclosure and create a risk of fire.
- Oxygen supports combustion. Oxygen must not be used while smoking or in the presence of an open flame. Only use oxygen in well ventilated rooms.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration varies, depending on the pressure settings, patient breathing pattern, mask, point of application and leak rate.
- The technical specifications of the mask are provided for your clinician to check if it is compatible with the flow generator. If it is used beyond technical specifications or used with incompatible devices, the seal and comfort of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect the function of the flow generator.
- Stop using the Mask and consult your physician or sleep therapist, if you have ANY adverse reaction to the use of the mask.
- Refer to your CPAP or bilevel device manual for details on settings and operational information.
- Remove all packaging before using the mask.

Components of the Mask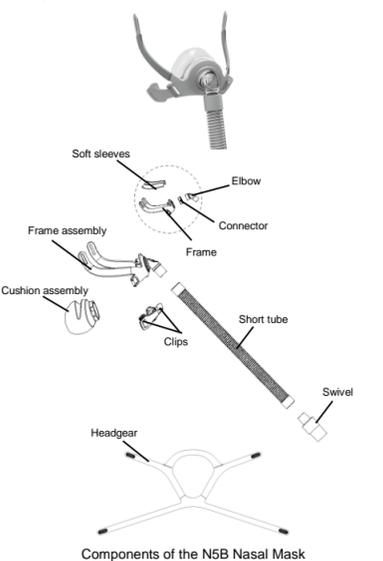


Fitting the Mask

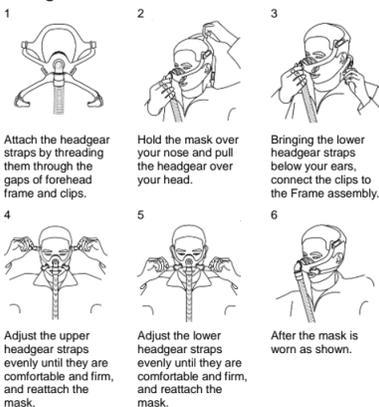Disassembling the Mask

Note:
The Frame assembly cannot be disintegrated.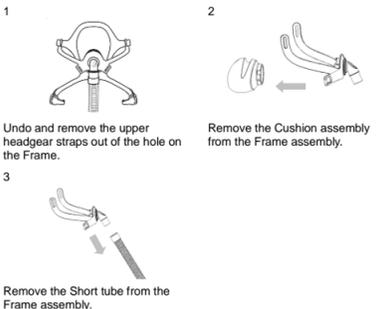

Cleaning the Mask at Home

Notes:

- The mask and headgear can only be cleaned through hand washing.
- The headgear may be washed without removing the clips.
- When washing the mask, the Elbow and the Frame cannot be disassembled from the Frame assembly.

⚠️ CAUTION

- Always follow cleaning instructions and do not use solutions containing bleach, chlorine, alcohol, aromatics, moisturizers, antibacterial agents, or scented oils to clean any part of the system or air tubing, otherwise may damage the mask.
- Exposing any part of the system or tubing to direct sunlight or heat may cause deterioration.
- If any visible deterioration of a component is apparent (cracking, crazing, tears etc), the component should be discarded and replaced.

Daily / After Each Use

- Disassemble the mask components according to the disassembly instructions.
- Thoroughly clean the separated mask components (excluding headgear), by gently rubbing in warm (approx. 30°C) water using mild liquid detergent (e.g., Alconox diluted at 1%) for up to 10 min.
- Use a soft bristle brush to clean the vent.
- Rinse all components well with drinking quality water and allow to air dry out of direct sunlight.
- When all components are dry, reassemble according to the reassembly instructions.


Weekly

Handwash the headgear and all components in warm (approx. 30°C) water using mild liquid detergent (e.g., Alconox diluted at 1%). Rinse the components well and allow them to air dry out of direct sunlight.

Reprocessing the Mask between Patients

This mask should be reprocessed when used between patients. Please refer to the accompanying "Cleaning and Disinfection Instructions" or BMC website www.bmc-medical.com for "Cleaning and Disinfection Instructions". If you do not have internet access, please contact your BMC representative.

Reassembling the Mask

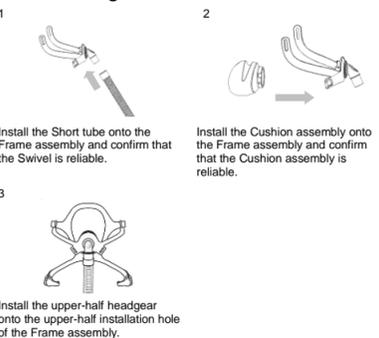Troubleshooting

Problem	Possible Reason	Possible Solution
Mask won't seal properly or is uncomfortable.	Mask may have been fitted incorrectly or is incorrectly adjusted.	Carefully follow instructions in "Fitting the Mask" section. Make sure the headgear is not over-tightened.
	Mask size is wrong.	Consult your clinician.
	Mask is not tightened correctly.	Tighten both upper and lower headgear straps.
	Cushion is misaligned or creased on face.	Ensure cushion is properly aligned according to the instructions in "Reassembling the Mask". Make sure cushion has no creases in it.
	Mask cushion may be dirty.	Clean cushion according to instructions in "Cleaning the Mask at Home".
Mask is too noisy.	Mask is assembled incorrectly.	Reassemble mask.
	Vents are blocked or partially blocked.	Clean the vents according to instructions in "Cleaning the Mask at Home" section.

Technical Specifications

Pressure-Flow Curve	Mask Pressure (cmH ₂ O)	Pressure (cmH ₂ O)	Flow Rate (L/min)
	4	4	19
	9	9	30
	15	15	39
	20	20	46
	25	25	52

Dead space is the empty volume of the mask up to the swivel. The dead space of the mask varies according to cushion sizes but is less than 105 mL.

Therapy Pressure 4 to 25 cmH₂O.

Resistance Drop in Pressure measured (average for 3 sizes) at 50 L/min: 0.2 cmH₂O
at 100 L/min: 0.8 cmH₂O

Sound DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level of the mask is 28 dBA, with uncertainty 3 dBA. The A-weighted sound pressure level of the mask at a distance of 1 m is 20 dBA, with uncertainty 3 dBA.

Environmental Conditions Operating temperature: +5°C to +40°C (41°F to 104°F)
Operating humidity: 0% – 93% relative humidity non-condensing
Storage and transport: -20°C to +55°C (-4°F to 131°F)
Storage and transport humidity: 0% – 93% relative humidity non-condensing

Storage

Ensure that the mask is thoroughly clean and dry before storing it for any length of time. Store the mask in a dry place out of direct sunlight.

Disposal

The N5B Nasal Mask does not contain any hazardous substances and may be disposed of with your normal household refuse.

Symbols

System and Packaging

	Caution, consult accompanying documents. Indicates a Warning or Caution and alerts you to a possible injury or explains special measures for the safe and effective use of the device.
	Lot number
	Catalog number
	Not made with natural rubber latex
	Temperature limitation
	Humidity limitation
	Manufacturer
	Authorized Representative in the European Community

Limited Warranty

It is warranted that N5B Nasal Mask, including its Frame assembly, Cushion assembly, Swivel, Clips and Headgear shall be free from defects in material and workmanship for a period of ninety (90) days from the date of purchase by the initial consumer.

To exercise the rights under this warranty, contact the local authorized dealers.

The expected service life of N5B Nasal Mask is one year.

MANUFACTURER:
BMC MEDICAL CO., LTD.
Room 110 Tower A Fengyu Building, No. 115 Fucheng Road, Haidian, 100036 Beijing, PEOPLE'S REPUBLIC OF CHINA
URL: en.bmc-medical.com
E-mail: intl@bmc-medical.com
Tel: +86-10-51663880
Fax: +86-10-51663880 Ext. 810

EU AUTHORISED REPRESENTATIVE:
Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße 80, 20537 Hamburg, Germany
Tel: 0049-40-2513175
Fax: 0049-40-255726

Manufacturing Site:
BMC (Tianjin) Medical Co., Ltd.
3/F, Building No.4, No.1 Xinxing Road, Wuqing District, 301700 Tianjin, PEOPLE'S REPUBLIC OF CHINA
Tel: +86-22-82939881

ITHALATCI: ECE TIBBİ CİHAZLAR VE MEDİKAL SAN. TİC. A.Ş Sümer Mahallesi Prof. Dr. Turan Güneş Cad. The Istanbul Vellefendi Sit. G1 Ticaret Apt. No: 57 AC Zeytinburnu – İstanbul - TÜRKİYE	TEKNİK SERVİS: Teviz Mah. Bakır Katlıcınece Yolu Sok. No:20/4 Zeytinburnu / İSTANBUL
MERKEZ: Sümer Mah. Prof. Dr. Turan Güneş Cad. No: 57 AC Zeytinburnu / İSTANBUL	MALTEPE: Başbuğ Mah. Başbuğluk Cad. No:10/2 Maltepe / İSTANBUL
SİRKETİ: Hacıpaşa Mah. Ebussuud Cad. Telli Öğü İşi Merkez No:30/4 Sirket – Fatih	ANKARA: Bahçelievler Mah. Sevkât Sunayra Aydemir Sok. No:4/4 Bahçelievler / ANKARA
KARAGÜL: Molla Gürani Mah. Turgut Özal Millet Cad. Karagül İş Merkezi No: 84/2E25 Fatih / İSTANBUL	BURSA: Cumhuriyet Mah. Z. Sabuncu Sok. No:32/A Osmangazi / BURSA
SİBAL: Zeynep Mah. Hızır Sokun T/2 Fatih / İSTANBUL (İSK Cibali Binası Yanı)	

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AB YETKİLİ TEMSİLCİSİ: