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MANUAL: Thank you for purchasing our product line, in order to make our products to better serve you, please read the instructions carefully before use and retain, to facilitate future maintenance and warranty.

Ultrasonic HUMIDIFIER

Instruction Manual

Fashion • LOVELY • Healthy

EXPERTS SUGGEST

Use appropriate and white vinegar or Banlangen can effectively remove the harmful bacteria in the air, and the prevention of influenza virus.

4. Please don't be metal goods, chemicals, such as detergent into the sink, otherwise, will appear the bad atomizing phenomenon.
5. The water in the water tank for a long time, that don't move, otherwise, the overflow of water transported machine will produce fault.
6. Please don't be placed in bright or air humidifier directly irradiated.
7. Don't be humidifier on the air, and as far as possible from the furniture and electrical products.
8. Please don't directly to the body flume water.
9. Is not a long time, please cut off the power, pull the plug.
10. Must not be to do with power to dump out the water remaining in the water, otherwise it sill immediately burned transducer.
11. When filled with water can be used continuously for 7-12 hours, not need of adding water.

Safety precautions

1. Please do not use wet hand to pull the plug, so as to avoid electric shock hazard.
2. Body tilt, please cut off the power supply, for further processing.
3. Start, don't touch the transducer, otherwise, will make you pain.
4. Did not cut off the power before, should not be decomposed body, do as to avoid the electric shock accident.
5. That no unauthorized replacement of power line.

Common fault treatment

Abnormal phenomenon	Reason	Processing method
Indicator light does not shine no wind, no fog	At the end of open power switch	Plug the power plug Turn on the power switch
Bright lights, there is no fog wind	To regulate the switch open	To adjust fog quantity switch
Spray fog has peculiar smell	The new machine The water tank is not clean	Open water tank, water tanks will be open In the shade 12 hours The cleaning tank, the replacement of a tank of water
Bright lights, no wind, no fog	Water tank water level too high	Will pour out some water in the sink, tighten the cover.
Spray a small amount	Transducer scaling Dirty water or storing for a long time	Cleaning transducer Replacement of the cleaning water

Product structure

Basic parameters

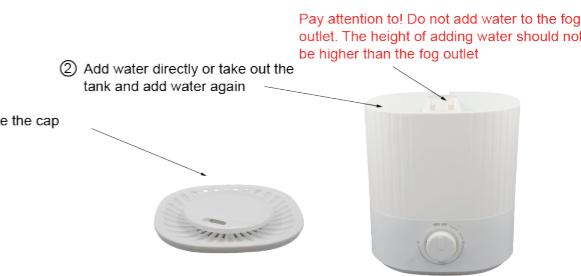
Rated voltage: 220V~ 50Hz
Rated power: 25W
Tank capacity: 2.5L
Maximum evaporation: 180-200ML/H
Noise: ≤35dB

Simple operation

Place the machine horizontally in a suitable place, take out the top cover, pick up the water tank and take out the sponge pad under the float of the machine base. Add water to the machine base directly or lift the water tank to add water, inject pure water or clean tap water. Connected to the AC 220V/50Hz power supply, the power light is "red", indicating that the water tank is filling the water tank. When the "green" light is on, it indicates that the water in the water tank is full and the machine can work normally. At this time, turn the fog amount adjustment knob to change the amount of spray, which can be adjusted according to the dry condition of the room. When the water tank is not water, the built-in installation device will automatically disconnect the atomization circuit. At this time, the mist exits without fog. At the same time the red indicator lights up. If you continue to use it, turn off the power and refill the water tank. (as shown in Figure 2 and Figure 3)

Operation of adding water

Operation note

1. Must not be to the water tank and the tank is added with 40°C above the water, in order to avoid discoloration or deformation.
2. Do not let water from the outlet flow of human bldy. The outside of the body of water, use a cloth to clean the.
3. In the indoor temperature at 0°C dry circle, please will sink and water in the water poured out, so that the water freezes, result in damaging components.

WARRANTY CARD

Dear customer:

Thank you for your purchase of the company's products, in order to protect your rights, please keep the warranty card, such as quality issues please contact the local dealer.

Name: _____ Model: _____

Purchasing address: _____

Telephone: _____ Date of purchase: _____

Address: _____

Matters needing attention

- 1) since the date of purchase please users enjoy one year repair service, please directly with the dealer to negotiate.
- 2) other parts such as the consumption of materials not under warranty.
- 3) repair, please present this card and proof of purchase or invoices effective.

Non warranty

- 1) Careless use or error caused harm.
- 2) the warranty card is lost or damaged.

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www.respirox.com.tr

Maintenance and cleaning

Before cleaning, please be sure to cut off the power, pulling a plug.

Tank cleaning (two to three weeks time)

Unscrew the cap, just a soft cloth to clean the tank man scale, with Shimizu Kiyoshi.

Tank cleaning aid (Monday times)

If the tank scale, use a soft cloth dipped in cleaning agent or white vinegar wash, and rinse with water.

External cleaning humidifier

The flexible cloth is below 40°C warm rinse, dry surface stains, the spray nozzle can wash water.

The humidifier collection

The need to clean, dry, returned to the original packaging and stored in dry place.

Warning

Never use the sharp apparatus to touch oscillating transducer, can use soft clean cloth.

Cannot be used with chemical solvents, petrol, kerosene to clean surface.

Don't use detergent to clean. The best use of clean water.

When cleaning, to avoid the water in the water, in order to avoid internal electrical parts wet, malfunction.

Do not use removable body component.